## ENGLISH

# MORROW BROWN® ALLERGY PRICK TEST NEEDLE

### INSTRUCTIONS FOR USE

The Morrow Brown® Allergy Prick Test Needle is manufactured by Riverside Medical Packaging Company Ltd (UK).

### INTENDED USE

Morrow Brown® Allergy Prick Test Needle is a Class IIa single use medical device. The needle is for the purpose of pricking a drop of allergen extract into the skin. The objective is to find out if the skin will react to the extract. A reaction would suggest a possible cause of the allergic problems with which the patient is suffering.

### INSTRUCTIONS FOR USE

1. Test sites should be at least 2.5cm apart.
2. Place a drop of extract beside each test site.
3. Peel open pack and remove the product by holding the flat end.  
**BE CAREFUL NOT TO TOUCH OR DAMAGE THE NEEDLE POINTS.**
4. By holding a single needle leg between thumb and forefinger, break off from the main product.
5. Firmly press the single needle point through the drop of extract.
6. Discard the used needle into an appropriate sharps container.
7. It is advised that a 'control' (without allergen) test site is undertaken.
8. Usually reactions can be read after 15-20 minutes.  
**DO NOT USE A NEEDLE MORE THAN ONCE.**

### STERILITY

The Morrow Brown® Allergy Prick Test Needle is supplied sterile in compliance with ISO11137 requirements for medical device to be designated "STERILE".

The product has been sterilized by Gamma irradiation.

### STERILE INFORMATION

- The integrity of the packaging for each device should be checked carefully for damage or contamination to ensure that the sterility and safety have not been compromised.
- The product is for single use only.
- The product cannot be re-sterilized.
- The product remains sterile until packaging is opened.
- Sterility of the product cannot be guaranteed beyond the expiry date.

### STORAGE

The product must be stored at room temperature with normal humidity conditions. Keep packaging dry and store out of direct sunlight.

### DISPOSAL

Used and un-used exposed (open pack) Morrow Brown® Allergy Prick Test Needle should be disposed of in an appropriate sharps container to prevent accidental needling of others.

### GENERAL WARNINGS

- Do not use if current date exceeds label expiry.
- Only for use by a qualified allergy specialist.
- Use only as instructed here-in.
- Do not use for anything other than the intended purpose.
- Under no circumstance must a needle be re-used; this may result in cross contamination of the patients or a false positive reaction. Each needle has been designed for single use only.

### CONTRA INDICATIONS

Allergy to needle material.

### POSSIBLE ADVERSE EFFECTS

Foreign body reaction to the probe caused by hypersensitivity to the plastic.

### RESIGNATION OF GUARANTEE

Riverside Medical Packaging Company Ltd accepts no responsibility for claims deriving from the misuse of the product.

### SYMBOL DEFINITIONS

European Conformity	Product sterilized by gamma irradiation
Do not use if package is damaged	Do not re-use
Do not re-sterilize	Catalogue number
Batch code of product	Use by date of product
Consult instructions for use	Minimum and maximum recommended storage temperature
Keep Dry	Keep away from sunlight
Manufactured by	Indicates the authorized European representative
United Kingdom Conformity Assessed	Single Sterile Barrier Symbol

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Riverside Medical Packaging Co. Ltd. Newmarket Drive, Derby, UK, DE24 8SW
Meddev Solutions Ltd